

DIP

NEW!

Test for Alcohol as low as 0.02% BAC

# STAT<sup>+</sup>DIP



## STATDIP<sup>®</sup>

The STATDIP<sup>®</sup> One-Step Dip Card is a simple and economical method to test for drugs of abuse. See the back for full procedure details and how easy it is to read results.

- Fast
- Accurate
- Reliable
- Easy-to-Use
- FDA approved
- Affordable

## CALL FOR A FREE SAMPLE!

### TEST FEATURES

- Simple Dip & Read Procedure
- Available with Built-in Adulterants
- Test Up to 15 Drugs (see below)
- "Kit" Packaging Available (includes collection cups & gloves)

### Drugs of Abuse:

Marijuana  
 Cocaine  
 Opiates  
 Methamphetamine  
 Amphetamine  
 Phencyclidine  
 Benzodiazepines  
 Barbiturates  
 Methadone  
 Ecstasy  
 Tricyclic Antidepressants  
 Oxycodone  
 Propoxyphene\*  
 Buprenorphine\*  
 6-Monoacetylmorphine\*

### Adulterants:

pH (acidic additives)  
 Specific Gravity (sample dilution)  
 Oxidants (bleach/oxidizing agents)

\*For Forensic Use Only




Toll Free: 866-933-0964  
[www.micro-distributing.com](http://www.micro-distributing.com)

# STAT<sup>+</sup>DIP

# EASY-TO-USE FAST ACCURATE

CALL FOR A FREE SAMPLE!

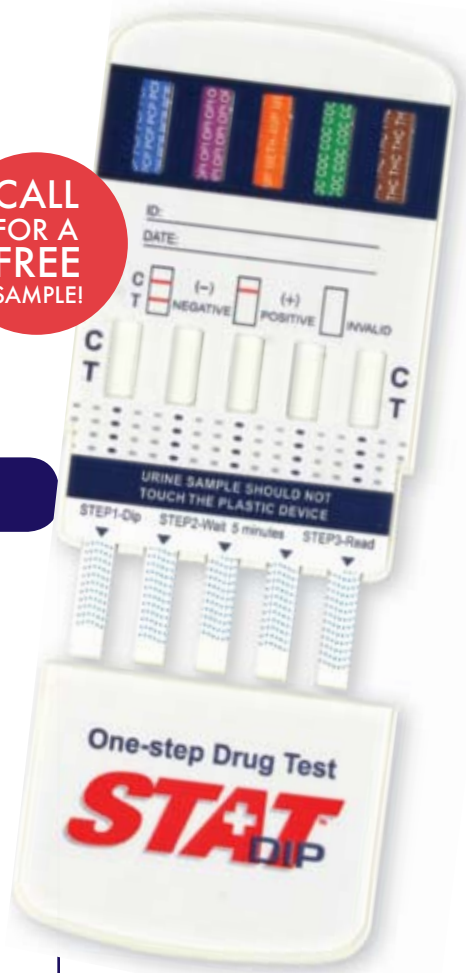

## PROCEDURE

### STEP 1

- Label the device with donor or control identifications. Remove the cap from the test device.



### STEP 2

- Immerse the absorbent tip into the urine sample about 10-15 seconds until the migration of urine fluid is observed in the result windows. Urine sample should not touch the plastic device.




### STEP 3

- Read results at 5 minutes. Results are stable for 60 minutes.



## INTERPRETING RESULTS

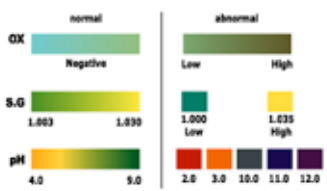

Interpret adulterant test strip between 3-5 minutes. Refer to enclosed color chart for an accurate interpretation.

Toll Free: 866-933-0964

Phone: 254-939-8923  
Fax: 254-939-5867  
sales@micro-distributing.com

