

FEATURED PRODUCTS

STAT⁺TEST



STATTEST[®]

Designed with simplicity in mind, the STATTEST[®] Drugs of Abuse Screening Tests are easy-to-use, fast, accurate, and affordable. The STATTEST[®] product line is FDA approved, earned the CE mark, and is available in a variety of urine and oral fluid test devices. Cut costs and save time with the STATTEST[®] product line.

Review the procedure information on the back to see how easy-to-use these products are and how fast you will see results.

CALL FOR A FREE SAMPLE!

TESTS AVAILABLE

Drugs of Abuse:

- Marijuana
- Cocaine
- Opiates
- Methamphetamine
- Amphetamine
- Phencyclidine
- Benzodiazepines
- Barbiturates
- Methadone
- Ecstasy
- Tricyclic Antidepressants
- Oxycodone
- Propoxyphene*
- Buprenorphine*
- 6-Mono Acetylmorphine*

Adulterants:

- pH (acidic additives)
- Specific Gravity (sample dilution)
- Oxidants (bleach/oxidizing agents)

TEST TYPES AVAILABLE

Drugs of Abuse:

- Test Cups
- Dip Cards
- Cassettes
- Oral Fluid Devices*

Alcohol*:

- Saliva Test
- Breath Test
- Urine Test - **NEW!**

*For Forensic Use Only

MD


MICRO-DISTRIBUTING

Toll Free: 866-933-0964
www.micro-distributing.com

EASY-TO-USE • FAST • ACCURATE

STAT⁺ SWAB

1. Collect sample
2. Insert swab into vial
3. Twist & tighten cap
4. Peel label off
5. Read results at 10 minutes

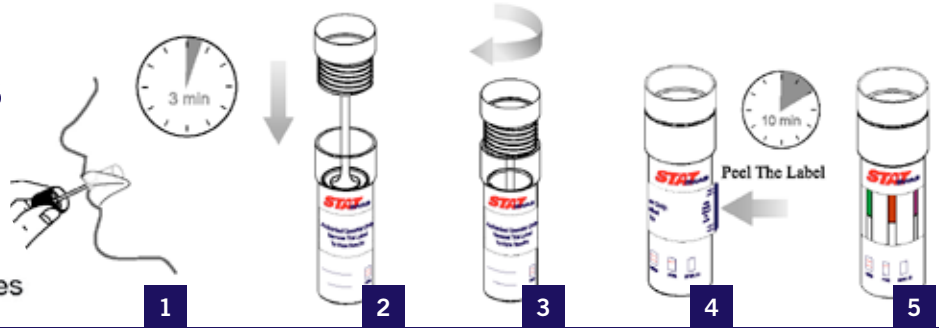


STAT⁺ CUP

1. Collect sample
2. Peel label off
3. Read results at 5 minutes




STAT⁺ DIP

1. Remove cap
2. Dip strips in urine for 15 seconds
3. Replace cap and lay flat. Read results at 5 minutes

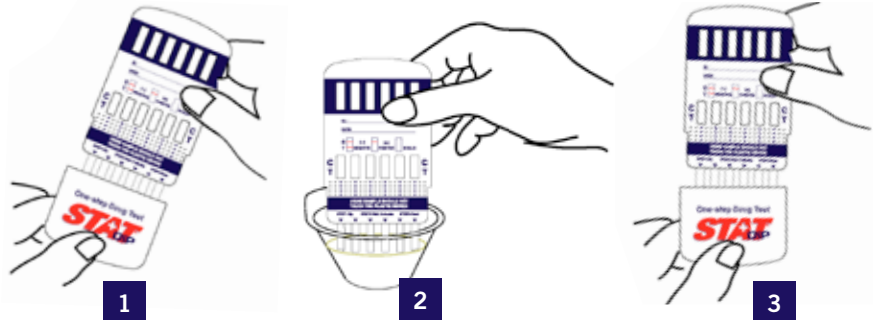


STAT⁺ CASSETTE

1. Use dropper to withdraw urine from collection cup
2. Dispense 3-4 drops in each sample well
3. Read results at 5 minutes

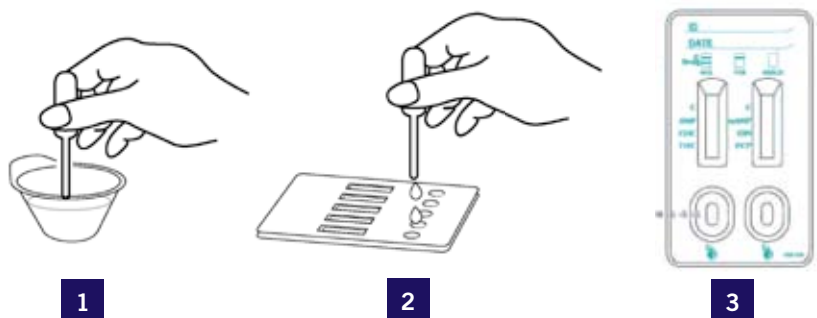

INTERPRETING RESULTS

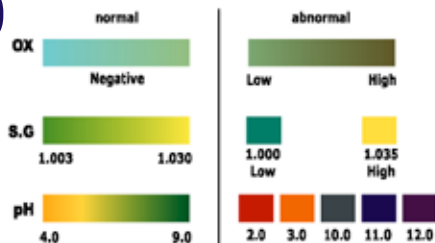

MD
MICRO-DISTRIBUTING

Phone: 254-939-8923
Fax: 254-939-5867
sales@micro-distributing.com