


Intect® 7 Instructions

Instructions:

Dip the test strip into the urine collection cup and remove immediately. Be sure all pads are wet.



Blot the test strip gently on its side to remove excess urine. Results are ready in one minute.

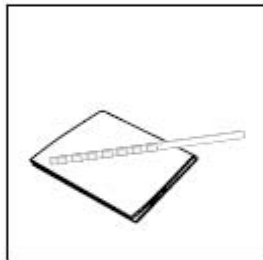

Results Interpretation: Compare the color of the test with the color chart and interpret. Results are void after 2 minutes.



Color Chart:

Intect® 7
Urine Test Strips for Adulteration Detection

See package insert for details
Für weitere Informationen siehe Packungsbeilage.
Vega l'inserto del pacchetto per i particolari.
Consultar el folleto interno para más información.
Se référer à la notice du coffret pour plus de détails.

Catalog No. / Katalog-Zahl / Numero di Catalogo / Referencia / Référence : AD007
25 test strips / teststreifen / dispositivi de analiza / tiras reactivas / bandelettes reactives

	Abnormal abnorm / anormale anormal / anormal		Normal normal / normale normal / normal			Abnormal abnorm / anormale anormal / anormal	
1. Creatinine (mg/dl) Kreatinin / Creatinina Creatinina / Creatinine	0	10	20	50	100		
2. Nitrite (mg/dl) Nitrit / Nitrit Nitrito / Nitrites			0	10		≥ 10	
3. Glutaraldehyde Glutaraldehyd / Glutaraldehyde Glutaraldehyde / Glutaraldehyde							
4. pH	2	3	4	5	10	>11	
5. Specific Gravity Spezifisches Gewicht Peso Specifico Gravedad Especifica / Densité	1.005	1.005	1.005	1.015		>1.005	
6. Bleach * Bleichmittel / Candeggina Lejía / Eau de Javel							
7. Pyridinium Chlorochromate * Pyridinium Chlorochromat / Piridinio Clorocromato Clorocromato de Piridinio / Pyridinium Chlorochromate							

* Presence of different color indicates other adulterants

Branan Medical Corporation Chella s.a.
10015 Mustangs Rd., Suite F 34, Av. De Iervunen, box 44, 1040 Brussels BELGIUM
Irvine, CA 92618 U.S.A. Tel: +32 2 732 59 54 Fax: +32 2 732 60 03

CE Patent No.:

Use color chart on bottle for most accurate color comparison

Reactions:

	Creatinine	Nitrite	Glutaraldehyde	pH	Specific Gravity	Bleach	PCC
Classic Adulterants							
Bleach		•			•	•	•
Drano					•	•	•
Vinegar					•		
Glutaraldehyde			•		•		
Sodium Nitrite		•			•	•	•
Commercial Adulterants							
PCC		•			•	•	•
Urine Luck 6.3							•
Stealth						•	•
Clean-X						•	•
Klear		•					
Instant Clean		•				•	•
Purafyzit						•	•
UR'n Kleen			•				
Dilution							
Urine Dilution	•				•		

Test Features:

- Accurate, most comprehensive test available
- Detects adulteration, dilution and substitution
- Simple to use and interpret
- Results in 1 minute
- No instrumentation required
- Conforms to DOT guidelines
- 18 month shelf life
- IntectCheck™ Controls available for monitoring performance