




# Multi-Panel Drug Test Cup

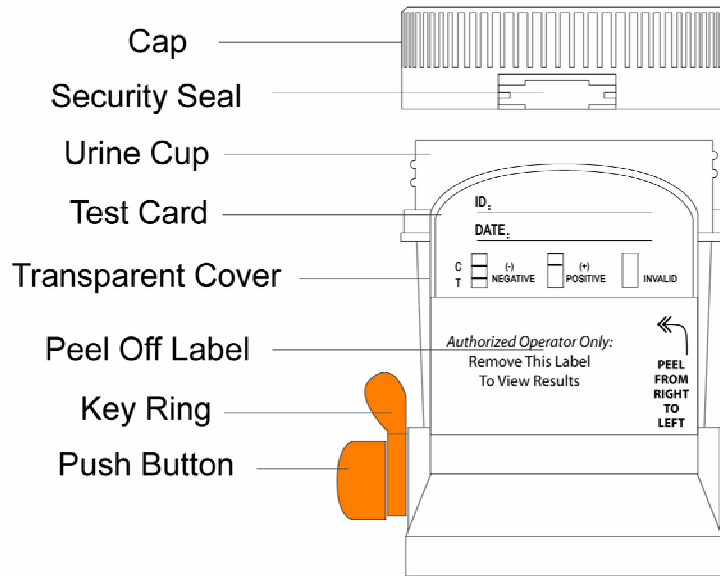


**1** Remove Cap.

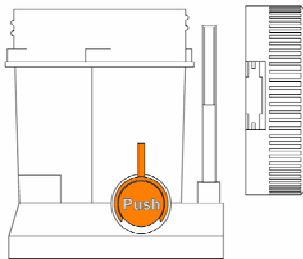


**2** Donor provides specimen.

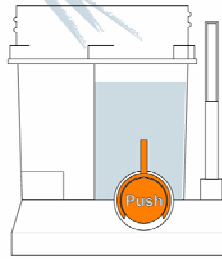


**3** Replace and secure cap. On a flat surface, remove the key ring and push the button.

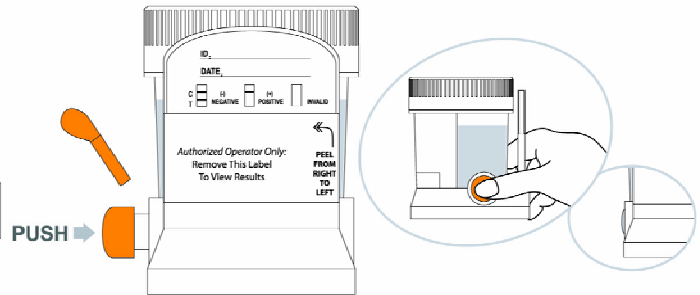

**4** Peel off the label to view results.

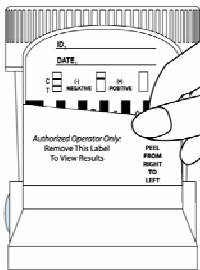


**5** Interpret results

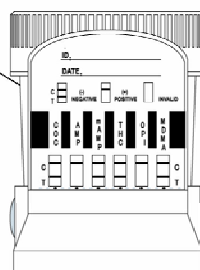

Test shows positive for Opiate and Amphetamine.



Negative results can be read as soon as they appear. Read positive results at 5 minutes.

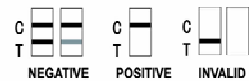

All positive results are presumptive and should be confirmed by an alternative method (e.g. GC/MS). A photocopy of the results can be made by placing the entire sealed cup with the card face down on the copier.