

CASSETTE

STATTM CASSETTE



STATCASSETTETM

The STATCASSETTETM is a reliable and efficient urine cassette drug test device. See the back for information on test procedure and how easy it is to interpret these test results.

**CALL FOR A
FREE SAMPLE!**

TEST FEATURES

- FDA Approved
- Simple Dropper Method
- Minimal Volume Required To Run Test
- Results in 1-2 minutes
- Test Up to 12 drugs
- Available with Built-in Specimen Validity Test
- Flat Face Allows for Photocopy of Results
- Preliminary Result Form Available



Toll Free: 866-933-0964
www.micro-distributing.com

STATTM CASSETTE

EASY-TO-USE

FAST

ACCURATE



CALL FOR A FREE SAMPLE!

PROCEDURE

STEP 1

- Label the device with donor identification. Using the specimen dropper, withdraw the urine sample from the specimen collection container.



STEP 2

- Slowly dispense 3~4 drops (approximately 120µL) into the sample wells (S), being careful not to overfill the absorbent pad.



INTERPRETING RESULTS

- Read results at 5 minutes. Results are stable for 60 minutes.



MD
MICRO-DISTRIBUTING

Phone: 254-939-8923

Fax: 254-939-5867

sales@micro-distributing.com

Toll Free: 866-933-0964