





Allow the kit and specimen to come to room temperature (65°F-86°F/18°C-30°C) prior to testing.
AVOID PLACING ANYTHING IN THE MOUTH 10 MINUTES PRIOR TO TESTING.


- 1 Remove the test cube and the sponge collector from the foil pouch by tearing at the notch. Place the test cube upright on a flat surface.

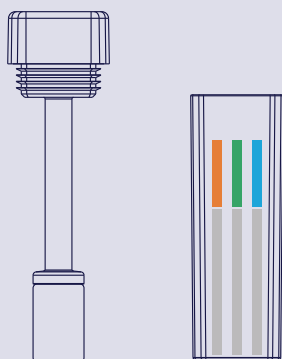

- 2 Put the sponge end of the collector in your mouth to collect oral fluid for about 7 minutes or until the sponge is fully saturated by oral fluid. Do not chew, bite or suck the sponge. If the amount of oral fluid does not make the sponge fully saturated within 7 minutes, repeat the collection using one additional sponge collector provided, beginning with Step 1.

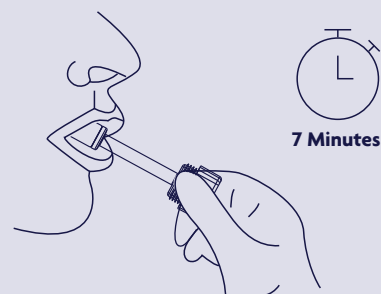

- 3 Place the fully saturated sponge collector inside the test cube. Press the sponge collector down firmly until the collector cap can no longer be pressed down. Keep test cube upright on flat surface and follow Step 4.
Note: Make sure the sponge collector is inserted vertically.

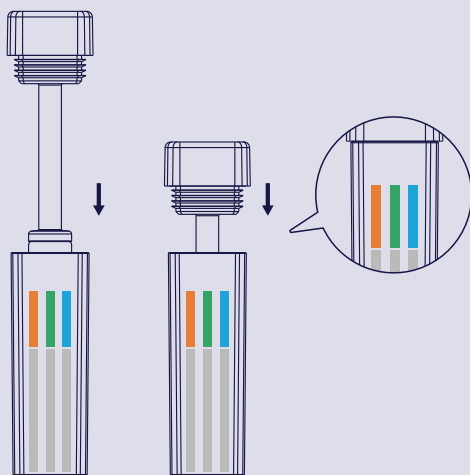

- 4 Interpreting Drug Test Results:
Read results at 5 minutes. Do not read after 5 minutes.

