

QUICK REFERENCE CARD

Multi-Drug Hair Test

Ensure all the supplies needed to perform the test are available. You will need the hair processing tube, hair test cube, and hair processor. (Make sure the hair processing tube and hair test cube have the same lot number)




1

Turn on the power switch on the back of hair processor.




2

Collect 30±5 (no more than 50) mg hair specimen and put it into the hair processing tube.

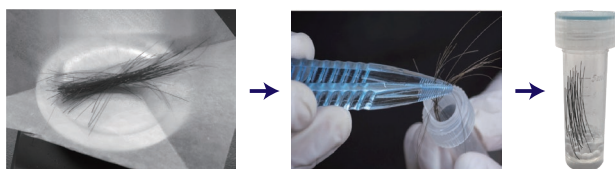

3

Place the hair processing tube containing the hair specimen into the hair processing tube holder. Press **Start**, the processor will run automatically for 2 minutes and 10 seconds.

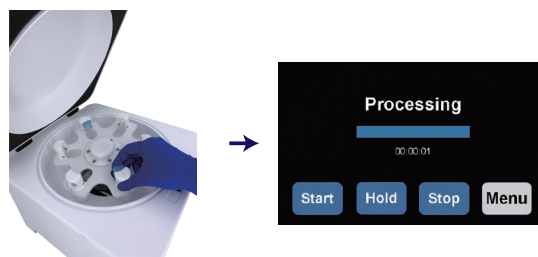

4


Note: When the processor runs for more than 0.5 hours, it is recommended to stop the operation and open the cover to dissipate heat for 15 minutes, as the temperature of the processor will rise to more than 185°F (85°C) after continuous operation for more than 1 hour.

Take out the hair processing tube and pour all the contents into the hair testing cube.



5

Read results at 5 minutes. Do not read after 10 minutes.

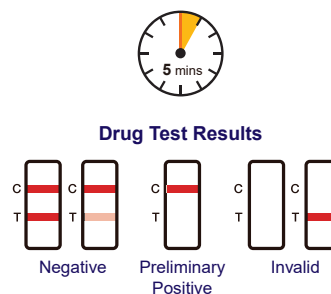

6