

STAT^{TEST}

AT HOME DRUG TEST

QUICK REFERENCE GUIDE


HOW TO USE THE TEST

1




Donor removes the cup lid and urinates directly into the test cup.
Recap the cup and place it on a flat surface.

2

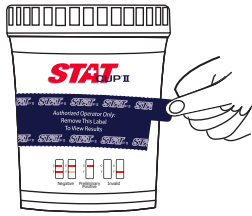

Peel off label to view results.

3

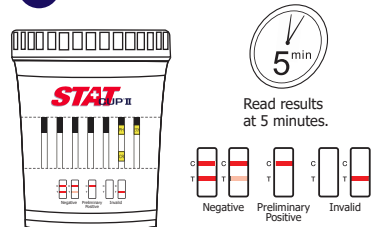

Read drug test strips at 5 minutes.

DRUG TEST STRIP ABBREVIATION KEY

THC	Marijuana
COE	Cocaine
WIC	Opiates
MEI	Methamphetamine
MEMA	Ecstasy
AMP	Amphetamine
PCP	Phencyclidine
BZO	Benzodiazepines
BAR	Barbiturates
MED	Methadone
OXY	Oxycodone
BUP	Buprenorphine

LABORATORY CONFIRMATION

Laboratory confirmation is a recommended but optional step for a preliminary positive result. If test is negative, there is no need to proceed.

1

- Confirm preliminary results
- Enter the Date of the Collection
- Check the drug(s) that were preliminary positive
- Affix top portion (barcode) to the back of the cup (don't cover the strips)

Place a check mark for the drug(s) that was preliminary positive.
 AMP BAR BUP BZO COC MDMA MET MOP
 MTD OXY PCP THC



ST051200305

Date of Collection: ___ / ___ / ___ (MM/DD/YYYY)

Please keep this portion in a safe place. You will need below confirmation number to access and receive your results.

CONFIRMATION NUMBER
ST051200305

For results: Call 1-866-933-0964
Or visit www.micro-distributing.com/hometest-results

2

- Keep bottom portion in a safe place
- Place the label on this form for safekeeping
- You will need the confirmation # to receive your result.



CUSTOMER SERVICE: 1-866-933-0964

LABORATORY CONFIRMATION

3

- Screw the cup lid on tight.
- Place the cup into the clear plastic specimen bag
- Place the bag into a shipping box

4

- Ship box to the Laboratory immediately via FEDEX, UPS, USPS, etc.
- Recommend Next Day delivery
- Samples not shipped within 24 hours should be refrigerated or frozen




Ship box to:
Cordant Health Solutions
2617 East L Street
Tacoma, WA 98421

PLEASE NOTE: Samples not received at the lab within 7 days won't be tested.

5

- Call for results at 866-933-0964 or view online at www.micro-distributing.com/hometest-results
- Enter your confirmation #
- Pay confirmation fee - \$25.00 via online or over the phone

EASY, ONLINE RESULTS

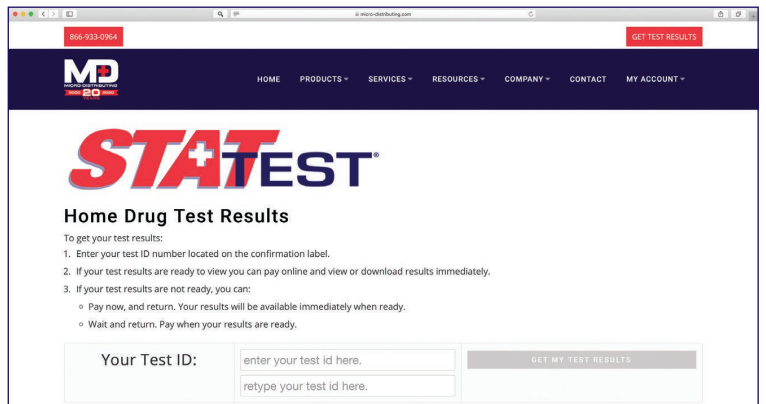

IMPORTANT INFORMATION

- Results not received within 7 days of the date of collection will not be tested
- Payment is required to receive confirmation results (\$25.00)
- A confirmation number is required to receive a result
- Results will be kept for 30 days only

SPEAK TO A DOCTOR ABOUT YOUR RESULT

A medical review officer (MRO) is a physician who is trained and certified to review drug test results.

They evaluate medical explanations (such as prescriptions) for confirmed positive, adulterated, substituted and invalid test results.

Follow the instructions online to pay for and access this service. Keep your confirmation number. A phone number is required for the physician to call you.

www.micro-distributing.com/hometest-results

SEEK HELP FROM A SUBSTANCE ABUSE PROFESSIONAL

The Substance Abuse and Mental Health Services Administration (SAMHSA) National Helpline is a free, confidential, 24/7/365 treatment and information service (in English and Spanish) for individuals and families facing substance abuse and mental health disorders.

This service provides referrals to local treatment facilities, support groups and community-based organizations.

1-800-662-HELP (4367)